

Assembly Guide



Part number: KDB799E01LY

Manufacturer: Zhongshan Kangdebao (KDB) Rehabilitation Equipment Co., Ltd.
Address: 24#, Longcheng Road, Shenglong village, Xiaolan town, Zhongshan City,
Guangdong Province, China

Preface

Thanks for choosing our product. This device is a shower & toilet aid for persons functional difficulties. Please read the instructions carefully before using. Please keep the instructions.

Warranty

1: The warranty period for this product will start from your purchase day.

Accessories: 90 days. Main frame: 365 days.

Please show your purchase receipt of this product when you claim warranty.

2: The following conditions void the free warranty.

- Modifying the structure of this product.
- Not using the product according to its intended purpose.
- The product is damaged or scratched by an unpredictable reason.

3: If you have any issues or suggestions, please contact our reseller or vendor immediately.

4. Notice: Before use, please check if any screw is loose or damaged.

To better protect the product, please clean with mild soap and warm water after use and wipe dry.

Record card

Booking number: _____

Name: _____

Phone number: _____


Purchased date: _____

Customer's address: _____


Parameters:

Total height	Total width	Total length	Tube diameter	Net weight	Weight capacity
71-83.Sc	53cm	51cm	2.2cm	4.6KG	125KG

Assembly Guide

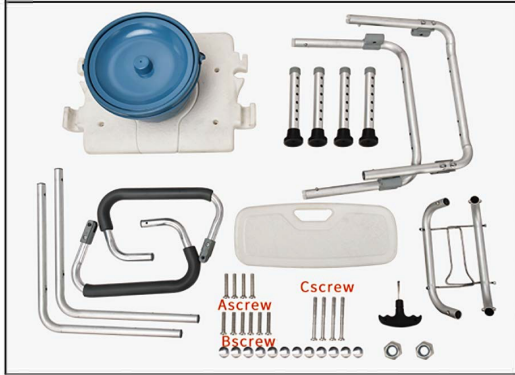

1. Components: Seat panel x1pc; Round bucket x1pc; Leg tube x4pcs; Seat frame tube x2pcs; Backrest tube x2pcs; Armrest x2pcs; Backrest x1pc; Side rack x2pcs; Hardware fittings: Screw A x4pcs; Screw B x6pcs; Screw C x4pcs; Arc metal washer x12pcs; Nut x2pcs; Screw driver x1pc.

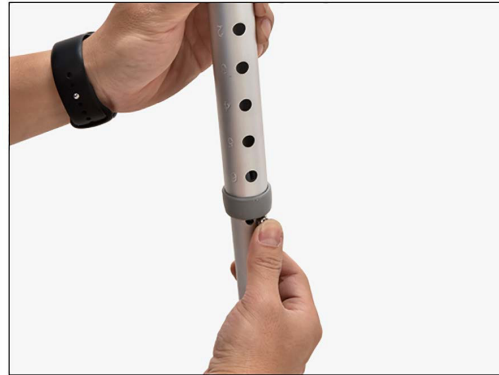

2. Attach the leg tube to the seat frame tube, press the marble slightly to insert the seat frame into the leg tube. Once inserted, make sure the marble pops out through the tube hole.



3. Align the seat frame tubes parallel and attach the side racks by inserting them into the plastic connector, make sure the marbles pop out completely. Larger sides of the racks must be placed in the back part of the seat



4. The seat frame installation is finished as shown in the picture.




5. Place the installed frame on the seat panel along the slot and press down slightly.



6. Attach the frame to the seat panel by using 4 screws and 4 arc metal washers. Use the 4 holes closer to the seat opening.



7. Attach the backrest tubes to the backrest with 4 screws B and 4 arc washers.

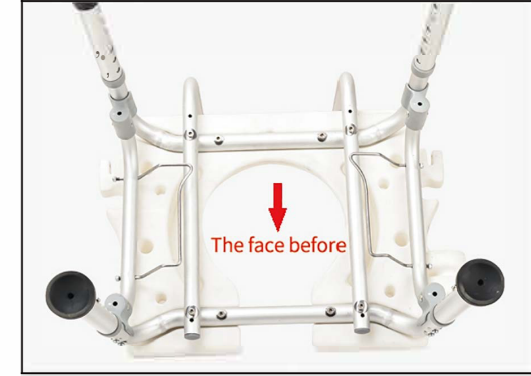

8. Attach the backrest to the seat panel (seat cut facing forward), use 4 screws C with 4 metal arc washers.



9. The backrest installation is finished as picture shows.



10. Insert a nut in the hole of clamp side in contact with seat. Attach armrest into the back slot of the seat panel and tighten it with B screw.



11. Installation is finished as shown in the picture.



12. The bucket can be removed by sliding it forward.